

Knowledge Article Template

◆ **Article Title: NON-RFID ITEMS (Supply Items) Quick Reference Cheat Sheet.**

◆ **Applicable Device Information:**

- **Device Type: Mire Prep, Mire Care, Mire ENTERPRISE SYSTEM**
 - **Software/Firmware Version (if applicable): any version**
-

◆ **Problem Description:**

Calls are made to internal support referencing supply items that may need to be added to the database for various tracking or formulary reasons. It can be vague to understand the clients' needs without having a reference point of what supply items are or how they are labeled and stored.

◆ **Common Symptoms:**

Client Calls to Support when leadership is not available to help add non-RFID items to the Enterprise Database.

◆ **Common Causes:**

- NON-RFID Items do not populate in RXNorm.
- Clients will want to track expiration date of supply items

◆ **Common Resolutions / Troubleshooting Steps:**

(Step-by-step instructions or resolutions. Include preventative actions if applicable.)

1. **Below is a list of common supply items.**
 - *Usually, clients will NOT TAG supplies and add them to non-RFID items for formulary build use.*
2. **Non-RFID items may not have standard Product Codes to import from RXNorm.**
3. **Customers may choose to tag and create “custom items” to track supply items, resulting in the need for a “Product Code”**
4. **Compare non-RFID item with Caller** – use this list to help compare supply items as needed to guide the client through using product code that is most relevant to their enterprise. (may require pharmacy leadership input. Pharmacy leader may alternate product code with REF (reference number) on the product if a 2D barcode is not available to scan).

5. **See Examples of Common Supply Items in the following pages:**

a) 0.9% Sodium Chloride Injection (Flush) 10mL (NS=Normal Saline = 0.9% Sodium Chloride)

Item Image:



SKU: 306546 GTIN: 00382903065462

BD PosiFlush™ Prefilled Normal Saline Flush Syringe, 10 mL Syringe 10 mL Saline Fill

[DISTRIBUTOR OPTIONS](#)

[CONTACT US](#)

Home > BD PosiFlush Normal Saline Filled Syringe ...



Flush box of 25

sodium chloride 0.9% in 10 ML Flush



Edit Item *Required

Item Information

Item Name: sodium chloride 0.9 % in 10 ML Prefilled Syringe CU: 727820

Alias: sodium chloride 0.9 % in 10 ML Flush Associate Additional CU's

Item Type: Medication Substance Type: Prefilled Syringe

Volume: 10 Select Unit* mL

Controlled Substance: Confirm* No

Product Codes Display Only My Items

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-ordering Days	Price	Billing Number	Action
00363-8071-00	Excelsior Medical Corporation	Controlled Room Temperature	Syringe	0100363807100114				
38290-3065-47	BD	Controlled Room Temperature	Syringe	0100382903065479				
38290-3065-53	BD	Controlled Room Temperature	Syringe	0100382903065530				
64253-0111-30	Medefill, Inc.	Controlled Room Temperature	Syringe	0110364253111300				
99999-1210-00	McKesson Corporation	Controlled Room Temperature	Syringe	0118006496800 121C				
36380-7100-11	Excelsior Medical Corporation	Controlled Room Temperature	Syringe	0100363807100015				
63807010011	Excelsior Medical Corporation	Controlled Room Temperature	Syringe	63807010011				
08290-3065-46	BD	Controlled Room Temperature	Syringe	0100382903065462				

Resource Link: <https://www.bd.com/en-us/products-and-solutions/products/product-page.306546>

b) 0.9% Sodium Chloride Injection (Flush) 3mL (NS=Normal Saline = 0.9% Sodium Chloride)

Item Image:



SKU: 306544 GTIN: 00382903065448

BD PosiFlush™ Prefilled Normal Saline Flush Syringe, 3 mL Syringe 3 mL Saline Fill

DISTRIBUTOR OPTIONS

CONTACT US

Edit Item *Required

Item Information

Item Name: sodium chloride 0.9 % in 3 ML Flush Item Type: Medication

Alias: Substance Type: Prefilled Syringe

Volume: 3 Select Unit* mL

Controlled Substance: Confirm* No

Concentration / Strength Formula Add Component

Name: Sodium cation Concentration / Strength: 0.9 Percent / 3 mL

Product Codes Add Product Code

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-ordering Days	Price	Billing Number	Action
82903-0654-04	BD	Controlled Room Temperature	Syringe	0100382903065448				
08881-5701-23	Covidien	Controlled Room Temperature	Syringe	0130884521005550				

Resource Link: <https://www.bd.com/en-us/products-and-solutions/products/product-page.306544>

c) Syringe – Luer Lock 10ml

Item Image:



SKU: 302995 GTIN: 00382903029952

10 mL BD Luer-Lok™ Syringe only

[DISTRIBUTOR OPTIONS](#)

[CONTACT](#)



Edit Item *Required

Item Information

Item Name: Syringe 10 mL Luer Lock Item Type: Medication

Alias: Substance Type: Select Substance*

Volume: 1 Controlled Substance: No

Concentration / Strength Formula

Name: Syringe Add Component

Concentration / Strength: 1 / Amount None Action

Product Codes Add Product Code

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-dating Days	Price	Billing Number	Action
08290-3029-95	BD	Controlled Room Temperature	Syringe	010038296302995211				<input type="button" value="Add"/> <input type="button" value="Remove"/>
46171-1000-02	Covidien	Controlled Room Temperature	Syringe	011088452117923311				<input type="button" value="Add"/> <input type="button" value="Remove"/>
REF302995	BD	Controlled Room Temperature	Syringe	302995				<input type="button" value="Add"/> <input type="button" value="Remove"/>

Resource Link: <https://www.bd.com/en-us/products-and-solutions/products/product-page.302995>

d) Syringe 1mL Tuberculin Safety

Item Image:

Home > Monoject - Tuberculin Syringe with Needle 1 mL 28 Gauge 1/2 Inch Regular Wall Sliding Safety Needle - 8881511201



Monoject - Tuberculin Syringe with Needle 1 mL 28 Gauge 1/2 Inch Regular Wall Sliding Safety Needle - 8881511201

by Cardinal
In Stock

\$231.99 USD

SKU 8881511201

**Spring Savings: Use code FALL for 5% OFF Total Purchase
Free Shipping Orders Over \$100**

This product may require 7-10 business days to process and prepare for shipping.

Resource Link: https://serfintymedical.com/products/kcm23-monoject-tuberculin-syringe-with-needle-1-ml-28-gauge-12-inch-regular-wall-sliding-safety-needle-8881511201-295803?variant=39943075790923&gad_source=1&gad_campaignid=19692553909&gbraid=0AAAAAC5zGAW56FwDWbUIywLbAFqGE8R09&gclid=EAIaIQobChMImZ_--eaDkAMVLEn_AR3JjxXyEAQYA_iABEGKIEvD_BwE or <https://www.cardinalhealth.com/en/product-solutions/medical/patient-care/sharps-safety/medication-delivery/safety-needles-and-syringes/monoject-insulin-and-tuberculin-safety-syringes.html>

e) 3mL Luer Lock Syringe

Item Image:

Edit Item *Required

Item Information

Item Name: 3mL Luer Lock Syringe Item Type: Medication

Alias: Substance Type: Select Substance*

Volume: 1 Select Unit* Count Clear Alias ⓘ

Controlled Substance: Confirm* No

Concentration / Strength Formula Add Component

Name: 3mL Luer Lock Syringe Concentration / Strength: 1 Select Unit* Count / Amount None Action

Product Codes Add Product Code

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-dating Days	Price	Billing Number	Action
08290-3096-57	BD	Controlled Room Temperature	Individually Wrapped	01003829030965721:				 
1180300777	Covidien	Controlled Room Temperature	Individually Wrapped	01108845210123011:				 
REF309657	BD	Controlled Room Temperature	Individually Wrapped	309657				 
REF309585	BD	Controlled Room Temperature	Individually Wrapped	0100382903095858				 

Resource Link: https://www.dotmed.com/listing/disposables-general/bd/309702/vacutainer-syringe-luer-lok-tip-3ml-300-casex/2764519?utm_source=base&utm_medium=search&utm_campaign=Base

f) Alcohol Swabs/Pads

Item Image:



How can we help you?



Change Country

Healthcare services ▾

Medical products and supplies ▾

Pharmaceutical products ▾

About us ▾

Careers ▾

Home / Professional products / Medical products and supplies / Skin and Wound Management / Medicated Wipes / Webcol™ Alcohol Prep Pads



Medicated Wipes

Medicated Wipes

Cardinal Health™ Curity™

Alcohol Preps

Webcol™ Alcohol Prep Pads



Webcol™ Medium and Large Alcohol Prep Pads

How to buy

Reach out to our team for ordering information.

Contact us

CUSTOMER SERVICE

For product specifications, service and certificate requests, please call our Customer Service team.

Call Customer Service 800.964.5227 »

Item Name

isopropyl alcohol 70 % Medicated Pad

Edit Item *Required

Item Information

Item Name: isopropyl alcohol 70 % Medicated Pad

Alias: [Clear Alias](#) ?

Item Type: Medication

Volume: Select Unit*

Controlled Substance: Control*

CUI: 797544 [Associate Additional CUI's](#)

Substance Type: Medicated Pad


Product Codes Display Only My Items

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-dating Days	Price	Billing Number	Action
53329-0805-09	Medline Industries, LP	Controlled Room Temperature	Individually Wrapped	MDS090735	<input type="text"/>	<input type="text"/>	<input type="text"/>	
63517-0500-02	Cardinal Health	Controlled Room Temperature	Individually Wrapped	5110	<input type="text"/>	<input type="text"/>	<input type="text"/>	

Resource Link: <https://www.cardinalhealth.com/en/product-solutions/medical/skin-and-wound-management/medicated-wipes/webcol-alcohol-preps.html>

g) 21G Needles

Item Image:

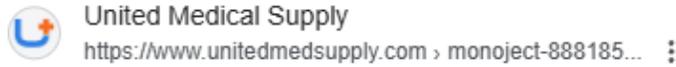

Monoject™ 8881850110 CRD50110

FEATURES · Easy-to-use, practical and intuitive one handed activation via multiple methods (finger, thumb or surface) for clinical flexibility · Even if bent ...



★★★★★

MONOJECT™ HYPODERMIC SAFETY NEEDLE, 21 GA, GREEN HUB, ...

NEEDLE SAF HYPODERMIC 21GA 1IN STERILE

Manufacturer: Cardinal Health

Brand Name: Monoject™

Part#: CRD50110

MPN: 8881850110

Min. Order Qty: 1

Qty. Interval: 1

Availability: Call for Availability

Edit Item *Required

Item Information

Item Name: 21G Needles Item Type: Medication

Alias: Substance Type: Select Substance*

Volume: 1 Select Unit: Count

Controlled Substance: No Confirm*

Concentration / Strength Formula Add Component

Name: 21G Needles Concentration / Strength: 1

Select Unit: Count / Amount None Action

Product Codes Add Product Code

Product Code*	Manufacturer*	Storage Type*	Package Type*	Barcode	Re-dating Days	Price	Billing Number	Action
08290309642	BD	Controlled Room Temperature	Individually Wrapped	01003829030516701				<input type="checkbox"/> <input type="checkbox"/>
88818-5011-50	Covidien	Controlled Room Temperature	Individually Wrapped	01108845210004761				<input type="checkbox"/> <input type="checkbox"/>
8881850110	Covidien	Controlled Room Temperature	Individually Wrapped	8881850110				<input type="checkbox"/> <input type="checkbox"/>

Cancel Save

Resource Link: <https://mms.mckesson.com/product/128989/BD-309642> or <https://www.unitedmedsupply.com/monoject-8881850110-crd50110>

h) bandages (individually wrapped)

Item Image:

Resource Link:

i) Gauze-Sterile (individually wrapped)




<https://www.cardinalhealth.com/en/product-solutions/medical/skin-and-wound-management/traditional-wound-care/woven-dressings/curity-sterile-gauze-pads.html>

g) Tray Lock Tags

Item Image:

Medication Safety and Pharmacy Products / Security Seals /

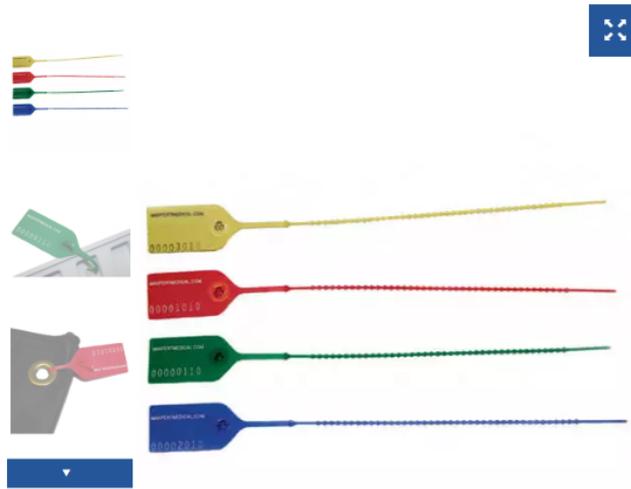

Adjustable Pull Tight Security Seal, Numbered

DTE03

This adjustable pull tight security seal or drug lock provides tamper indication for crash carts, crash cart trays, med kits, medical bags and boxes while providing visual indication when something has been opened or used.

Quantity: 100/pkg.

SKU	Color	Price	Quantity	Add To
DTE03B	Blue	\$12.00	<input type="text" value="1"/>	ADD TO
DTE03G	Green	\$12.00	<input type="text" value="1"/>	ADD TO
DTE03R	Red	\$12.00	<input type="text" value="1"/>	ADD TO
DTE03Y	Yellow	\$12.00	<input type="text" value="1"/>	ADD TO

Resource Link: <https://maxpertmedical.com/security-seals/adjustable-pull-tight-security-seal-numbered/>

h) Filter Micron 0.22 Image:



Links: <https://www.gilson.com/default/system-filter-for-macroman.html?srsltid=AfmBOoo4P1uBPgPIflVxBDnN50SPOMjJ0-geahZHgE3Wxq2yLiUvmUINQ60>

https://shop.cobetter.com/products/sterile-0-22-um-pes-syringe-filters-for-sterile-filtration?srsltid=AfmBOoq1amDIWY4JvCqFgboT2FsXDhI-FqaAn_uhkOUr1Vu6ZuouNrpt

g) Blunt Filter Needle (in kits that have glass ampoules)



Image:

BD® Blunt Filter Needles

- Are specifically designed to help reduce the risk of needlestick injuries during medication preparation*
- Features a filter with a 5 micron membrane that is designed to prevent large particles from being drawn into the syringe when preparing medications from ampoules
- Align to the 2024 INS Standards of Practice which recommend using a blunt filter needle when preparing medications from glass ampoules.²



Link: https://imedsales.com/bd-305211-blunt-fill-ndle-with-filter-18g-x-1-1-2inch-box-of-100-2?srsId=AfmBOorMZzV3F6aOrgi1M0hthjU_XukBDLIBkMKgdNmvf-j8qhaYhEVQKvA

◆ Escalation Criteria:

(When to escalate the issue to Development or Implementation team.)

- Issue persists after all troubleshooting steps have been attempted.
- Recurrent Tag(s) showing up (needs filtered out).
- Customer workflow is at a standstill.

◆ Related Documentation or References:

(Links or titles of user manuals, service bulletins, or SOPs.)

- Pharmacy Leadership
 - Troubleshooting Guide: Pharmacy Hardware
-

◆ **Last Updated By:**

- **Name/Role: Sara Thompson / Implementation Specialist**
- **Date: 10/03/2025**
- **Version: 1.0**