



How to Approve a Tray

1 Click "Approve Tray"

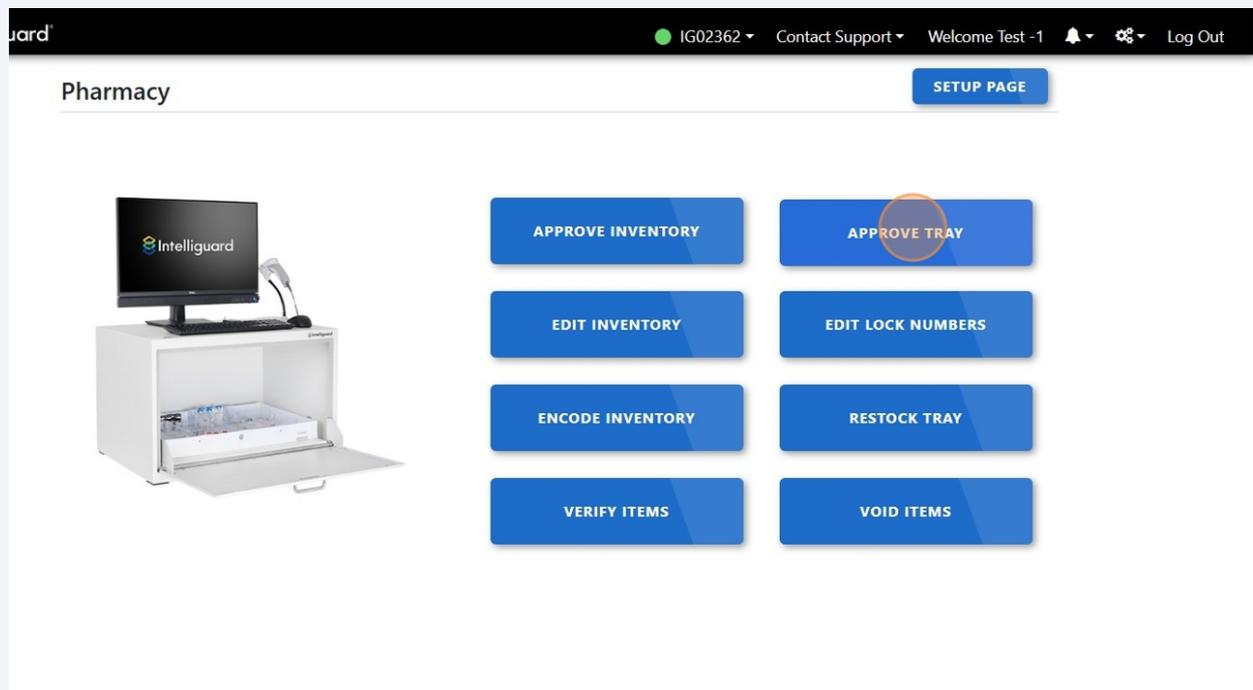

2 Place tray inside of the enclosure.

3 Click "Scan"

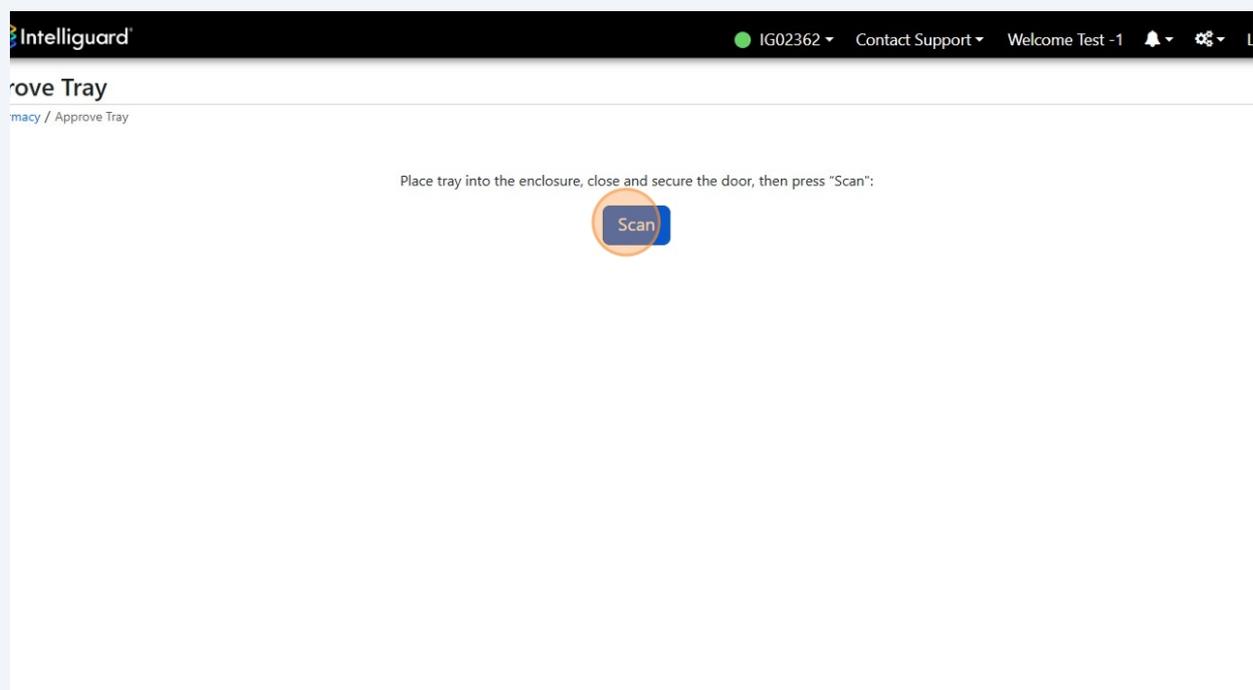

4 Visually approve the Item list that populates.

5 Click the "+" to see items expiring soon.

Name: Smart Path Room Adult Crash Cart Tray 01

Cycle: 4

Item List

Order ↑	Item Name	First Expiring Date	PAR Quantity	Current Quantity
	ceFAZolin 1 GM Inj	04/30/2026	5	5
	EPINEPHrine 1 MG in 10 ML Prefilled Syringe	09/30/2026	3	3
	esmolol HCl 100 MG in 10 ML Injection	10/31/2026	5	5
	etomidate 40 MG in 20 ML Injection	03/31/2026	2	2
	furosemide 40 MG in 4 ML	10/31/2026	6	6
	ketorolac tromethamine 15 MG in 1 ML Injection	12/31/2026	7	7
	Lidocaine 20mg/mL 5mL Inj	05/31/2026	2	2
	Naloxone 0.4mg/mL 1mL Inj	05/31/2026	2	2
	Ondansetron 2mg/mL 2mL Inj	11/30/2026	7	7

Expiring Soon

guard

6 Input the desired day amount and the list of expiring items will populate.

Name: Smart Path Room Adult Crash Cart Tray 01

Cycle: 4

Item List

Order ↑	Item Name	First Expiring Date	PAR Quantity	Current Quantity
	ceFAZolin 1 GM Inj	04/30/2026	5	5
	EPINEPHrine 1 MG in 10 ML Prefilled Syringe	09/30/2026	3	3
	esmolol HCl 100 MG in 10 ML Injection	10/31/2026	5	5
	etomidate 40 MG in 20 ML Injection	03/31/2026	2	2
	furosemide 40 MG in 4 ML	10/31/2026	6	6
	ketorolac tromethamine 15 MG in 1 ML Injection	12/31/2026	7	7
	Lidocaine 20mg/mL 5mL Inj	05/31/2026	2	2
	Naloxone 0.4mg/mL 1mL Inj	05/31/2026	2	2
	Ondansetron 2mg/mL 2mL Inj	11/30/2026	7	7

Expiring Soon

Expiring Within Days: 30

Item Name

Package Type

Expiring Date

Print Expiring List

guard

7 Click "Continue"

Intelliguard IG02362 Contact Support Welcome Test -1

Approve Tray

Pharmacy / Approve Tray

Scanned Tray Name: Smart Path Room Adult Crash Cart Tray 01

Continue

Complete Item List

Medications

Formulary Display Order ↑	Item Name	First Expiring Date	PAR Quantity	Current Quantity
1	ceFAZolin 1 GM Inj	04/30/2026	5	5
2	EPINEPHrine 1 MG in 10 ML Prefilled Syringe	09/30/2026	3	3
3	esmolol HCl 100 MG in 10 ML Injection	10/31/2026	5	5
4	etomidate 40 MG in 20 ML Injection	03/31/2026	2	2
5	furosemide 40 MG in 4 ML	10/31/2026	6	6
6	ketorolac tromethamine 15 MG in 1 ML Injection	12/31/2026	7	7
7	Lidocaine 20mg/mL 5mL Inj	05/31/2026	2	2
8	Naloxone 0.4mg/mL 1mL Inj	05/31/2026	2	2
9	Ondansetron 2mg/mL 2mL Inj	11/30/2026	7	7

RFID Items Expiring Soon

8 Here you can choose to enter a Lock Number for the approved tray.

Intelliguard IG02362 Contact Support Welcome Test -1

Approve Tray

Pharmacy / Approve Tray

Enter Lock Number:

Lock Number:

Approve

9

Click "Approve"

Guard

IG02362

Contact Support

Welcome Test -1



Log Out

ay

ve Tray

Number:

Lock Number:

1

Approve

